

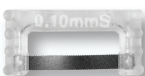



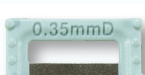



The ContacEZ *IPR Strip System* is a safe and innovative interproximal reduction system that makes interproximal enamel stripping efficient and accurate without creating sharp corners or subgingival ledges. Designed for use with clear aligners and other orthodontic treatments, these flexible diamond strips curve and conform to the natural contours of the teeth to maximize patient comfort and safety.

- ▶ Ergonomic, single-handed design
- ▶ Provides optimal tactile control
- ▶ Will not remove excess enamel
- ▶ Will not cut lips or gums
- ▶ Prevents sharp corners or subgingival ledges
- ▶ Effective for anterior and posterior areas
- ▶ Central opening for maximum visual perception
- ▶ Saves chair-time

### IPR Strip System

	<b>Yellow Starter</b> • 0.07 mm Single-Sided • Extra-Fine Diamond • Serrated	REF32108		<b>Red Opener</b> • 0.12 mm Double-Sided • Medium Diamond	REF32408
	<b>Dark Blue Widener</b> • 0.15 mm Double-Sided • Coarse Diamond	REF32508		<b>Green Extra-Widener</b> • 0.20 mm Double-Sided • Coarse Diamond	REF32608

### IPR Plus Strips

	<b>0.10 mm Single-Sided Opener</b> Clear • Med-Fine Diamond • Serrated	REF32208		<b>0.15 mm Single-Sided Opener</b> Cyan • Medium-Coarse Diamond	REF32308
	<b>0.25 mm Double-Sided Widener</b> Purple • Coarse Diamond	REF32708		<b>0.30 mm Double-Sided Widener</b> Brown • Coarse Diamond	REF32808
	<b>0.35 mm Double-Sided Widener</b> Mint • Coarse Diamond	REF36608		<b>0.40 mm Double-Sided Widener</b> Aqua • Coarse Diamond	REF36708
	<b>0.45 mm Double-Sided Widener</b> Turquoise • Coarse Diamond	REF36808		<b>0.50 mm Double-Sided Widener</b> Teal • Coarse Diamond	REF36908

### Usage Guidelines & Technique

This technique is especially effective for anterior teeth and for case refinement. ContacEZ *IPR Strips* are autoclavable (**steam heat only, no dry heat, up to 270°F for 10 minutes at 30 psi max**) and reusable, and should be sterilized prior to the first use. The manufacturer recommends to use each strip up to 5 times for maximum efficiency and functionality.

- ① Hold the strip with your thumb and forefinger, insert it into the interproximal space, and gently pass back and forth buccolingually until the strip is passive. Do not use vertical force. Start with the thinnest diamond strip, and repeat with thicker strips until you have reduced (stripped) the correct amount of enamel according to the treatment plan.
- ② To ensure accurate measurements, polish the proximal surface using the ContacEZ *Gray Final Polishing Strip*.
- ③ Verify reduction achieved using the ContacEZ *Incremental Thickness Gauge*.
- ④ Pediatric dentists can use ContacEZ *IPR Strips* for the serial proximal reduction of primary molars.

For convenience when autoclaving and organizing, store strips in a ContacEZ Strip Tray

