


The **ContactEZ Restorative Strip System** is an innovative, precision dental strip system designed to achieve ideal proximal contact adjustment and complete marginal seating of crowns, veneers, inlays/onlays, and proximal contouring of composite fillings accurately with minimum time and effort.

- ▶ Single-handed, ergonomic design offers optimal tactile control and reduces hand fatigue.
- ▶ Eliminates the need to hold small restorations and use rotary instruments or articulating films.
- ▶ Central opening for enhanced visual perception and access for tools.
- ▶ Patient-friendly design prevents gagging and soft tissue irritation.
- ▶ Flexible strips conform to the natural contours of the teeth to avoid creating sharp corners
- ▶ Autoclavable and reusable.

① Adjust proximal contact

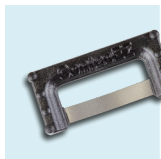

Black Diamond Strip
The Ideal Proximal Contact Adjuster
Fine Diamond • Single-Sided
REF 31108

Use the **Black Diamond Strip** to adjust ideal proximal contacts and restore interproximal relief (IR)* by eliminating excessive pressure between the indirect restoration and the adjacent teeth without opening the contact. Use this strip prior to definitive cementation of crowns, inlays/onlays, and veneers. It is also useful to contour proximal surfaces of composite fillings.



Orange Serrated Diamond Strip
The Proximal Contact Adjuster with Sawtooth
Extra-Fine Diamond • Single-Sided • Double-Serrated
REF 31208

Use this strip to adjust and contour proximal contacts for composite fillings. The serrated edge readily separates and penetrates tight contacts to prepare for ideal proximal contact adjustment. To clean out excess cement from the interproximal space, use the **White** or **Blue Serrated Strips**.

*The Journal of Prosthetic Dentistry, "A Proximal Contact Adjustment and Interproximal Relief Method" Vol. 97, No. 4, by Daniel S Kim, DDS & Kyu-Won Suh, DDS

② Clean out trapped cement



White Serrated Strip
The Gentle Saw, Interproximal Cement Cleaner
No Diamond • Double-Serrated
REF 31408

This ultra-thin serrated strip is designed to safely cut and clean out excess cement trapped in the interproximal space after cementation. Prior to light-curing, use it for interproximal cleaning of multiple veneers or composite fillings without disturbing placement and to separate them from adjacent teeth. Gently pass the strip back and forth buccolingually only.



Blue Serrated Strip
The Heavy-Duty Dental Saw
No Diamond • Double-Serrated
REF 31508

This heavy-duty dental saw is designed for more rigorous cutting and cleaning in the interproximal space. It safely cuts into massive excess resin cement trapped in the interproximal space as an alternative to the **White Serrated Strip**. It is also very useful for separating fused contacts of multiple veneers after light-curing.

③ Polish to a natural finish



Gray Final Polishing Strip
The Proximal Surface Polisher
Extra-Fine Diamond • Single-Sided
REF 31308

Use this strip to polish proximal contact surfaces of restorations and to restore a natural finish in one step. This strip also confirms interproximal relief (IR)* after definitive cementation. For interproximal stain removal and cleaning, use the **Hygienist Strip, Stain Remover**.



Patented
contacez.com

100% Guaranteed
800.537.8765



EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands



ContactEZ, LLC
217 SE 136th Ave, Suite 105
Vancouver, WA 98684
United States